

PT. Pillar Harmony

Rukan Malibu Blok H35/36
City Resort Residence, Jakarta Barat 11730
Telp. 021 - 2940 5822, Fax. 021 - 2940 5833
e-mail : pillarharmony@cbn.net.id



How To Use 10% Neutral buffered formalin

1. Volume ratio specimen / fixative = 1 : 50
2. The specimen thickness 1 cm max
3. Fixation time at room temperature : 15 – 25 °C
4. Fixation time 1 – 2 days
5. The product must be used exclusively by specialized technical operators.
6. Store the preparation at room temperature 15 – 25 °C.
7. Observe all state and local environmental regulations regarding waste disposal
8. If exposed or concerned wash with plenty of water and get medical advice
9. Do not use if the primary container is damaged.