

INSTRUCTION FOR USE

Bio Marking Dyes

Product code	Description	Size
05-000-030	30 ml bottle with dropper, 7 colours	7 x 30 ml
05-014-030	30 ml bottle with dropper, blue	8 x 30 ml
05-015-030	30 ml bottle with dropper, black	8 x 30 ml
05-016-030	30 ml bottle with dropper, green	8 x 30 ml
05-017-030	30 ml bottle with dropper, yellow	8 x 30 ml
05-018-030	30 ml bottle with dropper, orange	8 x 30 ml
05-019-030	30 ml bottle with dropper, red	8 x 30 ml
05-020-030	30 ml bottle with dropper, purple	8 x 30 ml
05-014-030-1	30 ml bottle with dropper, blue	1 x 30 ml
05-015-030-1	30 ml bottle with dropper, black	1 x 30 ml
05-016-030-1	30 ml bottle with dropper, green	1 x 30 ml
05-017-030-1	30 ml bottle with dropper, yellow	1 x 30 ml
05-018-030-1	30 ml bottle with dropper, orange	1 x 30 ml
05-019-030-1	30 ml bottle with dropper, red	1 x 30 ml
05-020-030-1	30 ml bottle with dropper, purple	1 x 30 ml
05-014-240	240 ml bottle, blue	1 x 240 ml
05-015-240	240 ml bottle, black	1 x 240 ml
05-016-240	240 ml bottle, green	1 x 240 ml
05-017-240	240 ml bottle, yellow	1 x 240 ml
05-018-240	240 ml bottle, orange	1 x 240 ml
05-019-240	240 ml bottle, red	1 x 240 ml
05-020-240	240 ml bottle, purple	1 x 240 ml

Expected aim

Product for the preparation of cyto-histological samples for optical microscopy.

Method

- 1.Shake the bottle before use;
- 2.For fresh specimens (not fixed yet), apply the dye on the tissue, directly from the bottle or with the help of a brush, let dry for 2-3 minutes and fix in formalin;
- 3.For fixed specimens, remove the excess of fixative from tissue surface and apply the dye like on fresh specimen;
- 4.The dyes bind to the tissue surface at room temperature (15-30°C) in 2-3 minutes;
- 5.At the end of the use, close the bottle to avoid evaporation.

Warning and Precaution

The product must be used exclusively by specialized technical operators.

Carefully read the information on the classification of dangerous substances on the label. Always refer to the safety data sheet where are available the information on the risk presented by the

mixture, the precautionary measures during use, the measures first aid and the event of accidental release.

Do not use if the primary container is damaged.

Contraindication

Obtain special instructions before use.

Wear protective gloves / clothing and eye / face protection.

Keep away from heat/sparks/open flames/hot surface. No Smoking.

IF ON SKIN (on hair): remove/take off immediately all contaminated clothing. Rinse skin with water/shower.