

INSTRUCTION FOR USE

Giemsa for Hp

<u>Catalog number</u>	<u>Size</u>
04-090803	75 test

Expected aim

Product for the preparation of cyto-histological samples for optical microscopy.

Method

- 1)Deparaffinise sections and bring to distilled water.
- 2)Preparation of buffer solution. Take out from bottle B 5 ml of solution and diluting 1:10: Utilize obtained solution for thepreparation of Giemsa working-solution.
- 3)Preparation of Giemsa working-solution. Take out 10 ml of reagent A and reach the volume of 40 ml with buffer solutionpreviously prepared.
- 4)Put the solution in a Coplin Jar and immerse sections for 30 minutes.
- 5)Drain the slide without washing and put the section for 15 seconds in reagent C.
- 6)Repeat step 5 with reagent D and E.
- 7)Clear in xylene and mount.

Warning and Precaution

The product must be used exclusively by specialized technical operators.

Carefully read the information on the classification of dangerous substances on the label. Always refer to the safety data sheet where are available the information on the risk presented by the mixture, the precautionary measures during use, the measures first aid and the event of accidental release.

Do not use if the primary container is damaged.

Contraindication

Obtain special instructions before use.

Wear protective gloves / clothing and eye / face protection.

Keep away from heat/sparks/open flames/hot surface. No Smoking.

IF ON SKIN (on hair): remove/take off immediately all contaminated clothing. Rinse skin with water/shower.