

INSTRUCTION FOR USE

Papanicolaou OG6

<u>Catalog number</u>	<u>Size</u>
05-12013	1 x 500 mL
05-12013/L	1 x 1 L
05-12013E	1 x 2,5 L
05-12013Q	4 x 2,5 L

Expected aim

Product for the preparation of gynecological specimens, urine cytology, fine needle specimens, sputum and bronchial washings, to be examined by optical microscopy.

Method :

For the execution of the staining method is required the use of reagent Papanicolaou EA50 and Papanicolaou Harris Hematoxylin.

1- Ethanol 95%	2 minutes
2- Distilled water	1 minute
3- Harris hematoxylin	2 minutes
4- Tap Water	1 minute
5- Ethanol 95%	15 seconds
6- OG6	2 minutes
7- Ethanol 95%	15 seconds (twice)
8- EA50	5 minutes
9- Ethanol 95%	15 seconds
10- Ethanol absolute	30 seconds (twice)
11- Xylene	2 minutes

Warning and Precaution

The product must be used exclusively by specialized technical operators.

Carefully read the information on the classification of dangerous substances on the label. Always refer to the safety data sheet where are available the information on the risk presented by the mixture, the precautionary measures during use, the measures first aid and the event of accidental release.

Do not use if the primary container is damaged.

Contraindication

Obtain special instructions before use.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

Wear protective gloves / clothing and eye / face protection.

Keep container tightly closed.

IF ON SKIN (or hair): take off immediately all contaminated clothing. Rinse skin with water/shower.