

INSTRUCTION FOR USE

Biodec R

Catalog number	Size
05-M03009	1 x 500 mL
05-03009Q	4 x 2,5 L

Expected aim

Product for the preparation of cyto-histological samples for optical microscopy.

Method :

- 1- Volume ratio specimen/decalcifier 1:100
- 2- Procedure time 6-12 hours <5mm thickness.
- 3- Post decalcifying procedure: reconditioning the sample. Rinse with slowly running water for 1 hour or apply 3 changes of PBS pH 7.4, 20 minutes each.

Warning and Precaution

The product must be used exclusively by specialized technical operators.
Carefully read the information on the classification of dangerous substances on the label. Always refer to the safety data sheet where are available the information on the risk presented by the mixture, the precautionary measures during use, the measures first aid and the event of accidental release.
Do not use if the primary container is damaged.

Contraindication

Obtain special instructions before use.
Wear protective gloves / clothing and eye / face protection.
Wash hands thoroughly after handling.
IF INHALED: Remove victim to fresh air and keep rest in a position comfortable to breathing.
Immediately call a POISON CENTER or doctor/physician.