

INSTRUCTION FOR USE

Harris Hematoxylin for Histology

Catalog number	Size
05-M06004	1 x 500 mL
05-06004/L	1 x 1 L
05-06004E	1 x 2,5 L
05-06004Q	4 x 2,5 L

Expected aim

Product for the preparation of cyto-histological samples for optical microscopy.

Method :

The Hematoxylin (nuclear staining) in combination with the Eosin staining (cytoplasmic staining) is the most used method in the histopathological routine.

- 1- Section to distilled water
- 2- Harris ' hematoxylin 5 minutes
- 3- Colour change in tap water (or scott water for 1 minutes) 5 minutes
- 4- 1% eosin (water solution) 5 minutes
- 5- Tap water 5 minutes
- 6- Dehydrate
- 7- Clearing agent and mount

Warning and Precaution

The product must be used exclusively by specialized technical operators.

Carefully read the information on the classification of dangerous substances on the label. Always refer to the safety data sheet where are available the information on the risk presented by the mixture, the precautionary measures during use, the measures first aid and the event of accidental release.

Do not use if the primary container is damaged.

Contraindication

Obtain special instructions before use.

Wear protective gloves / clothing and eye / face protection.

Wash hands thoroughly after handling.

IF ON SKIN: wash with plenty of soap and water.

Immediately call a POISON CENTER or doctor/physician.